

Acetocaustin®

Wart Treatment Pen

Carefully read the instructions for use prior to using Acetocaustin Wart Treatment Pen, even if you used the product before. The information may have been amended according to new research and findings in the time since you opened your last package. Keep these instructions for use in case you need to consult them again. For adults and children (4 years and over).

WHAT IS ACETOCAUSTIN WART TREATMENT PEN USED FOR?

Acetocaustin Wart Treatment Pen is a medical device for the treatment of common warts on hands and feet. Acetocaustin Wart Treatment Pen contains monochloroacetic acid, a substance which has a keratolytic effect on the skin. The wart will dissolve, layer by layer. Monochloroacetic acid ensures that the cells containing the wart virus are destroyed. The benefit of using Acetocaustin Wart Treatment Pen is that an application directly onto the wart does not hurt, which means it can also be used for children, when applied correctly by an adult.

Acetocaustin Wart Treatment Pen can only be used on common warts, characteristics are:

- Common warts primarily occur on hands and feet
- Same color as the skin
- Circumference varies from less than a millimetre to more than a centimetre
- Several warts usually occur simultaneously
- Contamination often occurs through direct contact, but also through towels or infected surfaces
- Scratching and nail biting increase the risk of contamination



Do not use Acetocaustin Wart Treatment Pen on senile warts

- Senile warts can come in different sizes and colors
- Average circumference is 0.5 to 1 cm
- Often start as a skin-colored, yellow brown, greyish-brown patch. Later on, they become bigger and fatter
- Appear as if they are lying on the surface of the skin and are easy to scratch off, but are actually firmly attached to the skin
- They can occur anywhere on the body, except for the palms of the hands and soles of the feet, and the lips. Most likely to occur on the chest, the back, the face, and the neck



Do not use Acetocaustin Wart Treatment Pen on plantar warts/verrucae:

- Plantar warts/verrucae are found on the bottom/soles of your feet
- Usually flat and surrounded by rough, thickened skin
- Resemble calluses
- Can contain black dots or tiny seeds



Instructions for use (Prevent burns):

Do's:

- Application by adults
- Use only on hands and feet. Do not use on other body parts
- Rub Vaseline or zinc ointment onto the surrounding skin to protect the skin
- Dab the wart just once when applying;
- Treat the wart only once per week as per how to use
- Allow the liquid to only come into contact with the wart
- Avoid contact with the healthy skin
- Allow the liquid to dry properly after applying
- Leave the wart uncovered
- The result is visible after 1 week
- Application to children should be performed by an adult
- Keep out of reach of children



Don'ts:

- Do not apply to warts on any part of the body other than the hands or feet
- Do not apply to any wart that is wholly or partially under a nail
- Do not apply on warts that occur completely or partially on the soles of the feet (plantar warts)
- Do not apply when your hands are shaking or your fingers are stiff
- Do not ab on wart more than once when applying
- Do not cover the wart after applying the liquid
- Do not apply on birthmarks, water warts, open wounds, hyperpigmentation, senile warts, plantar warts, dark-colored, hairy or weird looking warts
- Do not apply during pregnancy or lactation.
- Do not repeat treatment without a one-week interval.



⚠ CAUTIONS

Do not use Acetocaustin Wart Treatment Pen:

- Under the age of 4 years old
- If you are sensitive to one or more of the ingredients
- In the face (the fluid should never make contact with the eyes or the mucous membranes of the nose or mouth)
- Avoid contact with eyes. Contact with eyes may cause blindness
- Do not rub or touch eyes with hands if the product has been applied to them. If product gets into eyes, call medical help right away
- On genital warts or in the genital area
- On birthmarks, molluscum contagiosum, open wounds, moles, naevus, senile (brown) warts, plantar warts, darkly colored, hairy or strange looking warts
- On red, irritated, or infected warts
- Immediately after a surgical removal of warts, freezing of warts (cryotherapy) or a laser treatment. This particularly applies when the skin is red or painful after the treatment. In this case you will have to wait at least 4 weeks, until the redness or inflammation of the skin has fully disappeared. Only then you can start using Acetocaustin Wart Treatment Pen again
- During pregnancy or while breastfeeding
- On warts which occur completely or partially underneath the nail

- Do not use on warts that occur completely or partially on the soles of the feet (plantar warts)
- While having poor blood circulation: The treated area in diabetic patients and those with poor blood circulation may not heal as quick or completely

Be extra careful when using Acetocaustin Wart Treatment Pen:

- In case of doubt concerning the diagnosis (the change in the skin cannot be clearly explained as a wart)
- In case of immune deficiency (e.g. if you are using cytostatic medication)
- In case of diabetes, or other complaints such as vascular diseases, as wounds will heal less easily

In these cases, you may not use Acetocaustin Wart Treatment Pen before a doctor has been consulted.

⚠ WARNINGS

- Only intended for external use
- Use with utmost care. The fluid of Acetocaustin Wart Treatment Pen is corrosive
- If the healthy skin makes contact with the fluid, please rinse it with water. If, despite these precautions, the skin becomes irritated, it will have to undergo the same treatment as a burn
- Contact your doctor if your complaints do not subside
- Properly close the pen after use and store it in an upright position
- Keep out of reach of children
- Use by children, the elderly and less-abled is not recommended
- Avoid contact with clothes, textile, table-cloths or skin
- Stop using if blisters occur
- Treat wart no more than once and repeat only if necessary after a one-week interval
- The product should not be used beyond the expiration date
- The lot number and the expiration date appear on the pen as well as the outside of the outer packaging
- Do not use on multiple people as cross-contamination of the virus can occur
- Wash hands (except treated area) immediately after use

- Inspect carefully before each use. If the product appears damaged, stop using and throw it away
- Damaged product may leak fluid

SIDE EFFECTS

Acetocaustin Wart Treatment Pen may cause some side effects, although not everyone will experience these side effects. The side effects that could occur when using Acetocaustin Wart Treatment Pen are irritated skin, burning sensation, skin redness or oversensitivity reactions. If any of these side effects becomes serious, or if you notice any side effects not listed in these instructions for use, please consult your doctor or pharmacist.

INTERFERENCE WITH MEDICATIONS

No known interactions with other medication.

CONTENTS OF ACETOCAUSTIN WART TREATMENT PEN

A pen filled with a 3ml mixture of Monochloroacetic acid (50%) and purified water, Methylene blue.

STORAGE CONDITIONS

Store Acetocaustin Wart Treatment Pen at a room temperature of 15-25°C in the original container and out of reach of children.

The expiry date is stated on the bottom of the packaging.

Any serious incident that has occurred in relation to this product should be reported to the manufacturer and to the competent authority of the Member State in which the user is established.

For more information about Acetocaustin Wart Treatment Pen visit www.acetocaustin.com. For all product enquiries, please contact the distributor.

EXPLANATION OF SYMBOLS

The warning signs and symbols on the packaging and instructions for use are essential to ensure that you use this product safely and correctly and to protect you and others from injury. The symbols are explained in the table below:

Symbol	Explanation
	Symbol for the manufacturer
	Symbol that indicates to consult the instructions for use
	Symbol for the distributor
	Symbol that indicates that caution is necessary when operating the device
	Symbol for the expiration date
	Symbol that indicates that the item is a medical device
	Warning: Corrosive substance
	Not for children aged 3 years and younger
	Symbol that indicates to keep this product out of reach of children
	Symbol that indicates that this product should not be used by pregnant women or while breastfeeding
	Symbol for the Lot number
	Symbol that indicates that the product should be stored between 15°C and 25°C
	Symbol that indicates the Unique Device Identifier

How to use Acetocaustin Wart Treatment Pen

(Application only by adults):

Please apply Acetocaustin Wart Treatment Pen with utmost care.

The solution is caustic. Be extra careful when applying it to parts of your hands or feet that are more difficult to reach. This particularly applies to people with limited mobility or in case of disturbed fine motor functions in the system (trembling hands, stiff fingers).



1. Before starting, protect the healthy skin around the wart by applying a thick layer of Vaseline or zinc ointment.
2. Hold the Acetocaustin Wart Treatment Pen in a slanted position, as if writing. Do not press or squeeze the pen.
3. Place the tip gently on the wart.
4. Apply the solution twice, each time for about 3 seconds (count slowly: 21, 22, 23).
5. Make sure the solution is applied only to the wart and not to surrounding skin.
6. After application, allow the solution to dry for about 5 minutes.
7. Once dry, carefully remove the Vaseline or ointment around the wart.
8. Close the pen tightly after use and store it securely, out of reach of children.
9. Do not cover the wart after application. Do not treat the same wart more than once per week.

The fluid will need a week to dissolve the top layer of the wart, so you will not see an immediate result after application. Depending on the size and thickness of the wart, it may be necessary to repeat the treatment with Acetocaustin Wart Treatment Pen after one week. The average treatment duration is around 5 weeks. Do not repeat treatment within a one-week interval.



Apply on common warts on hands and feet only

3ml

CE 0482

Fred. Roeskestraat 115,
1076 EE Amsterdam,
The Netherlands

Streetname 115,
1076 EE City,
Country

WARNING: PREVENT BURNS:
CONSULT INSTRUCTIONS FOR USE
BEFORE USING!