

# Acetocastin®

## Wart Removal Plaster

CAREFULLY READ THE INSTRUCTIONS FOR USE PRIOR TO USING ACETOCAUSTIN WART REMOVAL PLASTER, EVEN IF YOU USED THE PRODUCT BEFORE. THE INFORMATION MAY HAVE BEEN AMENDED ACCORDING TO NEW RESEARCH AND FINDINGS IN THE TIME SINCE YOU OPENED YOUR LAST PACKAGE. KEEP THESE INSTRUCTIONS FOR USE IN CASE YOU NEED TO CONSULT THEM AGAIN. FOR ADULTS AND CHILDREN (4 YEARS AND OVER)

### WHAT IS ACETOCAUSTIN WART REMOVER PLASTER USED FOR?

Acetocastin Wart Remover Plaster is a medical device for the treatment of warts and verrucas. Acetocastin Wart Remover Plaster contains 40% salicylic acid, a substance which has a keratolytic effect on the skin. The wart will dissolve, layer by layer. Salicylic acid ensures that the cells containing the wart virus are destroyed.

The benefit of using Acetocastin Wart Remover Plaster is that an application directly onto the wart does not hurt, which means it can also be used for children, when applied correctly by an adult.

### How to use: (Application only by adults)

- Soak the wart in warm water for 5 minutes before application. Clean and dry the skin before use, ensuring targeted area is free of creams and oils.
- Acetocastin Wart Removal Plaster should be cut to size "if needed" to fit only the affected skin area of wart then remove the release liner and apply.
- Leave in place until it starts to detach. After maximum 48 hours, remove the plaster.
- Soak the affected area in warm water for 5 minutes before removing loosened skin.
- Repeat this procedure as necessary for up to 14 days.

### CAUTIONS

#### Do not use Acetocastin Acid Wart Treatment:

- Under the age of 4 years old by medical supervision
- Do not use if you are a diabetic person, hypersensitive to salicylic acid or suffering from poor blood circulation.
- Pregnant women and children: only use the device with healthcare supervision.
- For genital warts or in the genital area.
- Should not be used in the vicinity of the eyes nor in deep wounds with narrow openings.
- Skin damage will occur if the healthy skin come in contact with salicylic acid.

### WARNINGS

- Only intended for external use
- Contact your doctor if your complaints do not subside
- Keep out of reach of young children
- Use by children, the elderly and less able must be supervised by an adult.

- Avoid contact with clothes, textile, table-cloths or skin
- Do not re-use.
- Dispose carefully in a manner preventing cross-contamination.

### SIDE EFFECTS

Acetocastin Wart Remover Plaster may cause some side effects, although not everyone will experience these side effects. The side effects that could occur when using Acetocastin Wart Remover Plaster are irritated skin, burning sensation, skin redness or oversensitivity reactions. If any of these side effects becomes serious, or if you notice any side effects not listed in these instructions for use, please consult your doctor or pharmacist.

### INTERFERENCE WITH MEDICATIONS


As far as we know, there are no interactions with any other medications.

### CONTENTS OF ACETOCAUSTIN WART REMOVER PLASTER

Acetocastin Wart Removal Plasters are made of Moleskin Fabric full coated with adhesive substrate containing 40% Salicylic acid. One packaging contains 20 Acetocastin Wart Removal Plasters.

### STORAGE CONDITIONS

Store Acetocastin Wart Removal Plasters at a temperature of 0-40°C in a dry and dark place in the original packaging and out of reach of young children.

 The expiry date is stated on the bottom of the packaging.

For more information about Acetocastin Wart Removal Plaster visit [www.acetocastin.com](http://www.acetocastin.com). For all product enquiries, please contact the distributor.

### MEDICAL DEVICE:

Acetocastin Wart Removal Plaster is a class IIa Medical Device according to the Directive 93/42/EEC.


### DISTRIBUTOR

THEOTCLAB  
Fred. Roeskestraat 115  
1076 EE Amsterdam  
The Netherlands

 **E C Rep Ltd,**  
5 Fitzwilliam Square East  
Dublin 2, D02 R744, Ireland





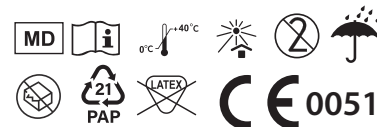




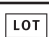
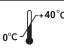



Acetocastin Wart Removal Plaster is exclusively produced by Pharmaplast s.a.e. for TheOTCLab:

 **Pharmaplast S.A.E**  
Amria, Free zone 23512  
Alexandria, Egypt



### EXPLANATION OF SYMBOLS

The warning signs and symbols on the packaging and instructions for use are essential to ensure that you use this product safely and correctly and to protect you and others from injury. The symbols are explained in the table below:

Symbol	Explanation
	Symbol for the manufacturer
	Symbol that indicates to consult the instructions for use
	Symbol that indicates that caution is necessary when operating the device
	Symbol for the expiration date
	Symbol that indicates that the item is a medical device
	Symbol indicates to keep the product away from sunlight.
	Indicates a medical device that is intended for one use.
	Keep out of reach of children
	Not for pregnant women or while breastfeeding
	Symbol for the Lot number
	The product shall be stored between 0 °C and 40 °C
	Indicates a medical device that should not be used if the package has been damaged or opened.
	Indicates the presence of natural rubber or dry natural rubber latex as a material of construction within the medical device or the packaging of a medical device which may cause allergic reactions.
	Authorized representative in the European Community / European Union